


A Safety notes

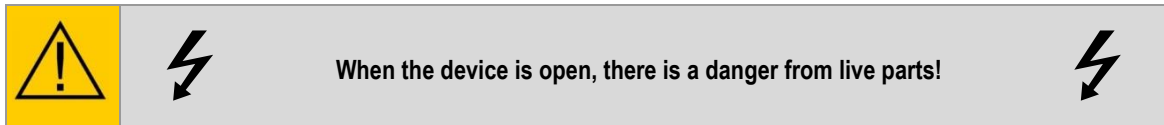

Please note that maintenance and repairs on h/p/cosmos products may only be carried out by h/p/cosmos authorized and qualified technicians.

Any damage caused by improper installation or installation not in accordance with the regulations is not covered by the warranty of h/p/cosmos.

B error message E01 appears

A1 definition

Error message E01 is purely an information message for checking the oil film under the running belt.
This message occurs every 1000km and may vary depending on usage and conditions.

A2 Reset oil message

Resetting the oil message works as follows:

<p>h/p/cosmos para control</p> <p>h/p/cosmos</p> <p>modus</p> <p>geschwindigkeit 0.0</p> <p>zeit 0:00</p> <p>index 0</p> <p>distanz 0.0</p> <p>steigung 0.0</p> <p>herzfrequenz pi 0</p> <p>geschwindigkeit</p> <p>steigung</p> <p>start eingabe</p> <p>STOP ESC</p> <p>1. modus auswählen (+ oder -) 2. start drücken</p>	<p>„+“ & „-“ & „Stop“ keys for approx. 4 sec.</p>
<p>h/p/cosmos para control</p> <p>h/p/cosmos</p> <p>modus</p> <p>geschwindigkeit OP:01</p> <p>zeit E. r E</p> <p>index S E t</p> <p>distanz</p> <p>steigung</p> <p>herzfrequenz pi</p> <p>geschwindigkeit</p> <p>steigung</p> <p>start eingabe</p> <p>STOP ESC</p> <p>1. modus auswählen (+ oder -) 2. start drücken</p>	<p>→ OP01" appears in the display Speed</p>

service instructions oil detection and belt lubrication

	<p>Confirm with "Start"</p>
	<p>➔ The display distance shows "done"</p>
	<p>Confirm with "Start"</p>
	<p>Exit the options with "Stop".</p>

A3 Procedure

If this message appears, do **not** automatically loosen the belt, but first check whether the oil under the belt is still sufficient.

To do this, first unplug the treadmill. Then reach under the belt with one hand and feel whether the oil is still sufficient. You should always feel a light lubricating film and it should not be "wet" or "dry".

- Is a light lubricating film can be felt: Do not oil
- To dry: please follow the point "Running belt lubrication" in these instructions
- Too wet: please contact h/p/cosmos. Further procedures will then be discussed.





service instructions oil detection and belt lubrication

D2 pluto g6

Needed material:

No.	amount	item number	item description
[01]	1	cos102443-01	lubrication set treadmill 150/50 LC

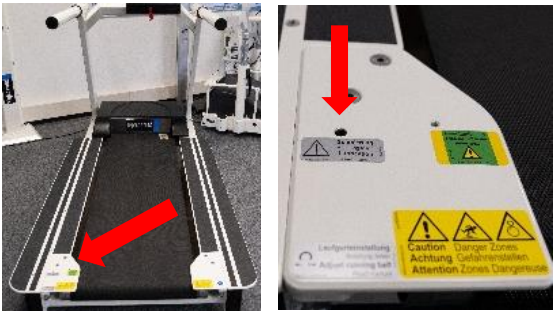Vorgehensweise

	Start the treadmill and let it run at a maximum speed of 5 km/h.
	Fill the syringe with 10ml of the special oil.
	Insert the syringe into the oil inlet
	Then fill the syringe with 10ml of air and pump it into the oil inlet so that no residual oil remains in the line.
	Allow the treadmill to continue running for at least 2 more minutes at a maximum of 5 km/h.
	Then walk on the tread at about 2 km/h, constantly changing your position to massage the oil on it.
	Clean excess oil from the area around the oil filler opening.

contact

We will be happy to assist you within our business hours:

(monday – friday from 08:00 am to 05:00 pm)

<https://www.hpcosmos.com/en/contact-support/contact>

h/p/cosmos sports & medical gmbh

Am Sportplatz 8

DE 83365 Nussdorf – Traunstein

Germany

phone: +49 86 69 86 42 0

direct service: +49 86 69 86 42 25

fax: +49 86 69 86 42 49

email: service@hpcosmos.com